

Easy ECG Monitor PC-80B User Manual



Instructions to User

Dear Users,

Thank you very much for purchasing our product. Please read the manual very carefully before using this device.

These instructions describe the operating procedures to be followed strictly. Failure to follow these instructions may cause measuring abnormality, device damage, or personal injury.

The Manual is published in English and we have the ultimate right to explain the Manual. No part of this manual may be photocopied, reproduced or translated into another language without the prior written consent. We reserve the right to improve and amend it at any time without prior notice.

Version of This Manual: Ver2.0

Revised date: May 14, 2019

All rights reserved.

3502-2430005

Announcement

Information furnished by the manufacturer is believed to be accurate and reliable. However, no responsibility is assumed by Creative for its use, or any infringements of patents or other rights of third parties that may result from its use.

Declaration of Conformity:

Creative hereby declares that this device complies with the following standards:

IEC 60601-1

IEC 60601-1-2

and follows the provisions of the Council Directive MDD 93/42/EEC.

For user's convenience, we share the latest version analysis software of this device on our website, the user can enter into our website (www.creative-sz.com/software/PCsoftware) to download the corresponding latest version data manager software. Please contact the manufacturer or your local distributor if anything about software downloading.

WARNING

1. Check the device to make sure that there is no visible damage that may affect user's safety and measurement performance. Stop using the unit, when there is obvious damage.
2. Do NOT make diagnosis oneself by the measurement and measurement results, always consult the doctor if abnormal information is presented frequently.
3. It is NOT designed or intended for medical diagnosis.

4. WARNING for PACEMAKER PATIENTS

Heart rate calculation may also count the pacemaker pulse for the patient who carries a cardiac pacemaker because of no pacemaker pulse rejection capability for the device. Do not rely entirely upon the heart rate reading of this device for pacemaker patients.

5. Do NOT use the device in the bathroom or moist circumstance.
6. Do NOT operate it in the environment where strong electro-magnetic interference exists.

7. The device is not equipped with detachable parts, the user should NOT take its shell apart without permission.
8. Connect/Disconnect data cable to the monitor gently and carefully, do NOT wring or shake the data cable plug with force to insert into or unplug from data interface port, or the data interface port may be damaged.
9. All the parts of the monitor should NOT be replaced at will. If necessary, please use the components provided by the manufacturer or those that are of the same model and standards as the accessories along with the monitor which are provided by the same factory, otherwise, negative effects concerning safety and biocompatibility etc. may be caused.
10. Necessary service must be performed by qualified service engineers ONLY.
11. In case of ECG leads damage or aging, please replace the lead.
12. Electric parts of electrodes, leads and cable are forbidden to contact any other conductive parts (including the ground).
13. Use only one type of electrode on the same patient to avoid variations in electrical resistance. It is strongly recommended to use silver/silver chloride

electrodes to ensure accurate measurement result.

14. When disposing of the device and its accessories, the local law should be followed.

Caution: U.S. federal law restricts this device to sale by or on the order of a physician.

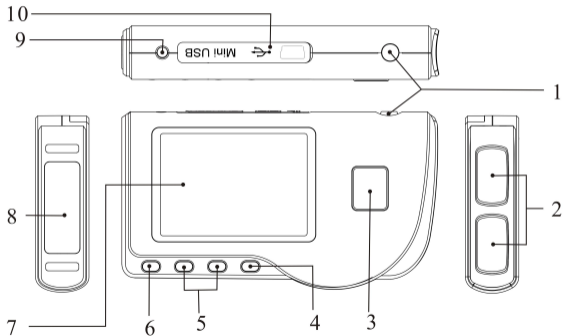
Table of Contents










1 OVERVIEW	1
1.1 APPEARANCE	1
1.2 NAME AND MODEL	3
1.3 STRUCTURE	3
1.4 FEATURES	3
1.5 INTENDED USE	4
1.6 OPERATING ENVIRONMENT	5
2 INSTALLATION	5
2.1 INSTALLATION OF BATTERIES	5
3 OPERATION	6
3.1 START TO MEASURE ECG	6
3.2 ECG MEASURING METHOD	10
3.3 ECG MEASURING PROCEDURE	15
3.4 WAVE REVIEW	22
3.5 DATA MANAGEMENT	33
3.6 SYSTEM SETTING	37


3.7 HELP.....	47
4 TECHNICAL SPECIFICATIONS.....	48
4.1 ECG MEASUREMENT.....	48
4.2 POWER.....	49
4.3 CLASSIFICATION.....	49
4.4 DISPLAY.....	50
4.5 DATA MEMORY.....	50
4.6 OVERALL DIMENSION AND WEIGHT.....	50
5 ACCESSORIES.....	51
6 MAINTENANCE AND SERVICE.....	52
6.1 MAINTENANCE.....	52
6.2 CLEANING AND DISINFECTING INSTRUCTION.....	53
7 TROUBLESHOOTING.....	54
8 ERROR MESSAGE AND SOLUTIONS.....	57
APPENDIX.....	59
GENERAL KNOWLEDGE OF ECG.....	59

1 Overview

1.1 Appearance



1.  **Power:** Long time press (about 2 seconds) to power on or off the device; short time press turns on/off the backlight.
2. **Metal Electrode I, II**
3.  **Measurement:** shortcut button for measuring, press this button to start measuring.
4.  **OK:** confirm selection or modification.
5. **Navigation key:**
 -  /  **Up/left/increase:** move the cursor upwards/to the left, or adjust parameter values.
 -  /  **Down/right/decrease:** move the cursor downwards/to the right, or adjust parameter values.
6.  **Return:** back to upper level of menu.
7. **Display screen:** display ECG waveforms and data.
8. **Metal Electrode III**
9.  **ECG Lead wire connector:** for connecting the lead wire.

10.  **Data interface:** for connecting Mini USB data cable.

1.2 Name and Model

Name: Easy ECG Monitor

Model: PC-80B

1.3 Structure

PC-80B Easy ECG Monitor consists of main board, panels and electrodes.

1.4 Features


1. Small in size, light in weight and easy to carry.
2. One-key measurement, easy to operate.
3. ECG waveform and interpretation of results are displayed clearly on dot-matrix LCD screen.
4. Seventeen types measurement result can be presented.
5. Up to 1200/2700 pieces of records for quick measurement, or 10-hour/22.5-hour data recording for continuous measurement, the data

storage depends on the provided size of the built-in memory.

6. Data records can be reviewed, copied, deleted, and uploaded as well.
7. Power saving technique with automatic shutting-down function.
8. Two AAA batteries are used to support more than 10 hours working time (wireless transmission function is disabled).
9. Wireless data upload function (optional).

1.5 Intended Use

PC-80B Easy ECG Monitor is intended for measuring and recording the ECG signal and average heart rate of adult patient. It is applicable for use in clinics and homes, and convenient to operate by the patients themselves.

 The device is not an ECG monitor as used in clinical institution or hospital, but is used for spot-check purpose ONLY. It can not be used to substitute the normal ECG examination or real time monitoring. The measurement results are a useful reference for doctor, but do not make direct diagnostic or analytical decision based on the information provided by this device.


1.6 Operating Environment

Operating temperature:	5~40°C
Operating humidity:	30%~80%
Atmospheric pressure:	70~106kPa

2 Installation

2.1 Installation of Batteries

1. Open the battery cover (see the Figure 2-1).
2. Insert two AAA size batteries directly.
3. Close the battery cover.

 Do NOT insert batteries with their polarities reversed. Disposing of the used batteries, the local law should be followed.

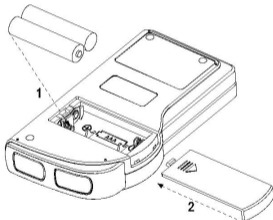


Figure 2-1

3 Operation

3.1 Start to measure ECG

Press the Power button (lasting for 2 seconds) to turn on the unit. The unit starts to initialize, and displays the following pictures Figure 3-1A and Figure 3-1B.

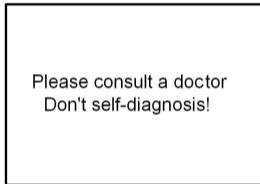


Figure 3-1A

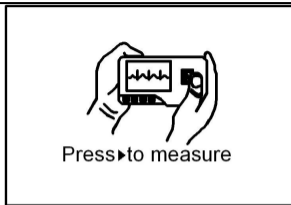




Figure 3-1B

After boot-strap, the unit enters menu screen, as shown in Figure 3-2.

Note: During initialization, press “ Return” button to enter main menu screen, as shown in Figure 3-2. If press “ Measurement”, it will enter measuring screen, as shown in Figure 3-4.

There are 6 tool buttons, press navigation key to shift cursor and press “■ OK” to confirm. Each functional button will be defined as follows:

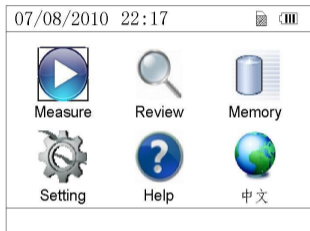


Figure 3-2



Measure: detect ECG waveform and HR value, and analyze whether the ECG waveform is normal.



Review: review the ECG records saved in this device (including ECG waveform and measurement result).



Memory: check the storage space and delete the ECG records saved in this device.



Setting: set parameters such as date, time, language, beeper, filter mode, brightness and wireless etc.



Help: provide the user with measuring methods etc. information.



中文: set the display language. This device is designed with dual-language: 中文(Chinese) and English, which can be switched alternatively by user.

3.2 ECG Measuring Method

3.2.1 Quick and Short ECG Measurement

1 Chest Measurement

In order to obtain ideal ECG signal, chest measurement is the first recommended method for it can ensure stable signal and fewer interference. Hold the device with the right hand. Make sure that the palm and fingers touch the metal electrode I and II reliably. Place the electrode III on bare skin about 5cm below the left nipple. The ECG signal detected by Chest Measurement is similar to Lead V ECG signal.



Figure 3-3A Chest Measurement

Possible improper operation:

- A. The contact place of electrode III deviates from left chest.
- B. The holding gesture is incorrect.
- C. The electrode can not contact bare skin completely. (For example place the

electrode on clothing.)

2 Leg Measurement

If it is not convenient to adopt chest measurement, the leg measurement will be recommended. When measuring, hold the device with the right hand. Make sure that the palm and fingers touch the metal electrode I and II reliably. Place the electrode III on the skin 10cm above the left foot ankle. The ECG signal detected by Leg Measurement is equivalent to Lead II ECG signal.



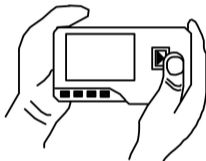
Figure 3-3B Leg Measurement

Possible improper operation:

- A. Wrong holding gesture.
- B. The electrode III can not contact bare skin completely. (For example, place the electrode III on the sleeve of trousers or socks.)

3 Palm Measurement

Palm Measurement is recommended in terms of convenient and quick measurement. Hold the device with right hand. Make sure that the palm and fingers touch the metal electrode I and II reliably. Press the electrode III closely against the center of the left palm. The ECG signal detected by Palm Measurement is equivalent to Lead I ECG signal.



3-3C Palm Measurement

Possible improper operation:

- A. Waving your both hands at will.
- B. Both hands loosely contact with electrodes during measurement.
- C. It is not palm contacts the electrode.

3.2.2 Continuous or Long-term ECG measurement

1 Lead Wire Measurement

To obtain clear and high quality ECG signal, the lead wire measurement can be used. Connect the lead wire firmly to the lead wire socket of the device. Place the electrodes and connect the lead wires as Figure 3-3D to obtain the Lead II ECG signal; if you want to measure Lead I and Lead III ECG signal, according to the following table to connect the lead wires to the electrodes.

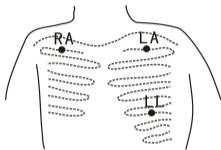


Figure 3-3D Lead Wire

Measurement

Possible improper operation:


- A. Wrong in connecting lead port.
- B. Place the electrode in the wrong part of your body.

Electrode	Lead	Lead I	Lead II	Lead III
Name				
Electrode Location				
The intersection between the centerline of the right clavicle and Rib 2.		R/RA	R/RA	L/LA
The intersection between the centerline of the left clavicle and Rib 2.		F/LL	L/LA	R/RA
Between the left edge of the breast bone and Rib 5		L/LA	F/LL	F/LL

ECG Leads Configuration and Electrodes Location Table

3.3 ECG Measuring Procedure

3.3.1 Quick Measurement

1. After choosing a proper measuring method as described in Chapter 3.2.1, press “ Measure” to enter quick ECG measurement automatically, as shown in Figure 3-4. The first 15 seconds is preparing phase during which the device will judge

whether the position of electrode is befitting or not according to measured ECG waveform. Please adjust your holding position if the detected waveform is not smooth.

On-screen display:

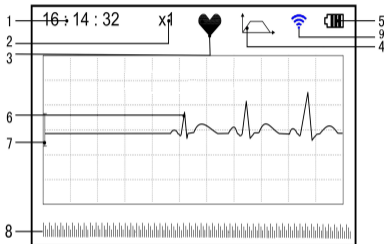

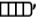





Figure 3-4 ECG Measuring Screen

- 1. **“16: 14: 32”**: Current time.
- 2. **“X1”**: ECG waveform gain.
“X1/2”: Waveform scaled with half of the basic gain;
“X1”: Waveform scaled with basic gain;
“X2”: Waveform scaled with twice of the basic gain
“X3”: Waveform scaled with three times of the basic gain
- 3. **“♥”**: Heart beat mark. Its flashes are synchronized with heart beats.
- 4. **“”**: Indicating that ECG filter mode is “Enhanced”; Refer to section 3.6.5 for filter setting.
- 5. **“”**: Status of battery power
- 6. **“”**: ECG waveform
- 7. **“|”**: Ruler for 1mV amplitude
- 8. **“”**: Compact compression of scaled waveform.

- 9. “”: Wireless mark (optional). When the “Wireless” is set as “ON” status in system setting screen, this mark will display on the screen. After a successful connection between the device and the remote host, the ECG data can be uploaded to the host (PC) for further recall, analysis and storage.

2. When the compact compression of scaled waveform reaches to the end, it means the measurement finishes, then the device steps into data analysis phase which is followed by measurement results display as shown in Figure 3-5.

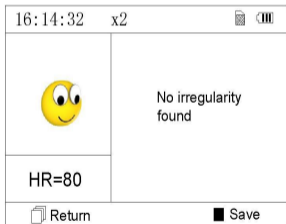




Figure 3-5 Measurement Result Screen



Figure 3-6 Data Saving

“ **Return**”: return to main menu screen directly and the current measurement result and measuring value won't be saved.

“ **Save**”: save the current measurement result and measuring value.

3. In the measurement result screen, the current measurement data will be saved in this device automatically without any operation in 6 seconds or directly press the “ **Save**” key to save the data, as shown in Figure 3-6. The device will return to the main screen automatically after the data is saved.

4. If the wireless transmission is selected, the device will go to the screen as shown in Figure 3-7 after the data is saved.

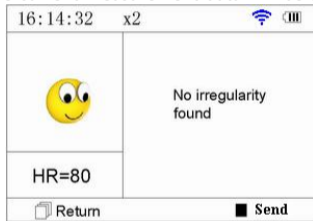






Figure 3-7

Press the “ **Send**” key on the device side to start transmitting ECG record wirelessly, and on the host side, the “Wireless Receive” icon should be clicked while running software “ECG Viewer Manager” on PC so that the data records can be

transmitted and saved to PC. After that the device will return to the main screen automatically. (Refer to *User Manual for "ECG Viewer Manager"* for detailed operation procedure)

3.3.2 Continuous Measurement

1. After choosing the continuous measuring method as described in Chapter 3.2.2, press the “ Measure” to perform continuous ECG measurement automatically as shown in Figure 3-8. The difference between the continuous and quick measurement is described below:

“ **Exit**”: the prompt information of operation on the screen. Press the “ **Return**” button to end the ECG measurement and return to the main screen. If the time of continuous measurement is over 30 seconds, this data will be saved automatically.

“**HR=59**”: indicates the real-time heart rate.

2. During the continuous measurement, if the wireless transmission function is




Figure 3-8

selected and a successful connection is setup, click the “Wireless Receive” icon on the host side while running software “ECG Viewer Manager” on PC so the data record can be transmitted and saved to PC (Refer to *User Manual for “ECG Viewer Manager”* for detailed operation procedure.). The data measured in wireless real-time transmission status won't be stored in the unit.

3. If there is no operation after the continuous measurement has been carried out for 3 minutes, the unit will adopt power-saving mode automatically (the screen will darken), but the measurement won't be interrupted. Once you press any button or the alarm event occurs, the system will exit power-saving mode, and the screen resume to normal brightness.

Notes: 1) Figure 3-5 shows a regular ECG waveform measurement result, this device can provide 17 types measurement results, refer to Table 3-1(Section 3.4.3) for details.

2) During measurement, if the measured part and metal electrodes loosely contact, the device will prompt “Contact?”, as shown in Figure 3-9. If there isn't signal, it will prompt “No signal”. The user can press “ **Return**” button to exit from the measurement.

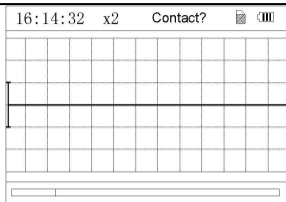


Figure 3-9

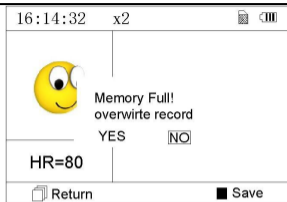



Figure 3-10

3) The device provides built-in 16MB/32MB memory clips (selecting “Memory” on main menu screen to see the detailed memory space), at most 1200/2700 data records for quick measurement or 10-hour/22.5-hour data records for continuous measurement can be saved in this device, and when the device connects to PC successfully, it works as a removable USB disk which contains four files, and 300/720 pieces of quick measurement can be recorded in each file.


4) When the storage space is full, the device will indicate and prompt that

“Memory full! Overwrite record?” with a display of “” mark as shown in Figure 3-10. Meanwhile, from now on, it won’t store the new record until the user manually deletes the unnecessary data or upload the records to PC to free some memory.


5) The device will power off automatically if there is no any key operation or signal inputting for a while.


3.4 Wave Review

3.4.1 Operation Description

1. Select “Review” on main menu screen, then press “ OK” button, the device enters record list screen as shown in Figure 3-11


Note:

“” : means the measured result of this record is regular.

“” : means the measured result of this record is irregular.

“” : means waveform with interference or lead off during measuring.



“”: If the enhanced mode is used, the icon of enhanced mode will be shown behind the record.


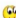
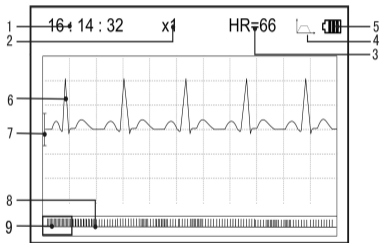
Records	1/32		
30/08/2012 09:37:22			
30/08/2012 09:25:21			
30/08/2012 08:34:22			
30/08/2012 07:37:12			
30/08 08:36:08–30/08 08:56:10			

Figure 3-11 Record List

NOTE: There is no such legend icon display for the record of continuous measurement. Only the start and end measuring time (not including the year) is displayed here.

2. Choose a record in record list, then press “■ OK” button to review this ECG record, the waveform display for quick measurement and continuous measurement are shown in Figure 3-12 and Figure 3-13 respectively. To stop auto replay, just press the Navigation key “◀/▲” or “▼/▶”. After that you can press the Navigation key “◀/▲” or “▼/▶” again to view the waveforms manually.



**Figure 3-12 ECG Data Review Screen
(Quick Measurement)**

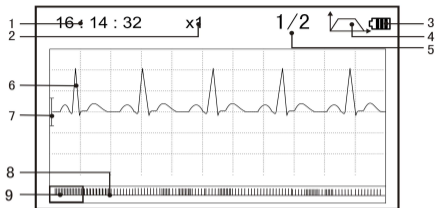







Figure 3-13 ECG Data Review Screen (Continuous Measurement)

On-screen display:

- 1. "16: 14: 32": ECG waveform measuring time.
- 2. "X1": ECG waveform gain. There are 4 options available:
 - "X1/2": Waveform scaled with half of the basic gain;
 - "X1": Waveform scaled with basic gain;

“X2”: Waveform scaled with twice of the basic gain;

“X3”: Waveform scaled with three times of the basic gain.

- 3. “HR=66”: heart rate value is 66bpm. For waveform review of continuous measurement, the heart rate value won't be displayed. “1/2”: the screen page description for waveform review of continuous measurement. “1”: current page, “2”: total page.
- 4. : indicates that ECG filter mode is “Enhanced”.
- 5. : Status of battery power.
- 6. : Reviewed ECG waveform.
- 7. “1”: Ruler for 1mV amplitude.
- 8. : Compact compression of scaled waveform.
- 9. : ECG waveform selecting box.

Note : ① the selected waveform in selecting box is displayed on the current screen.

② ECG waveform miniature in selecting box is the ECG waveform measured in 30 seconds.

3. Press “■ OK” button again, the measurement result of the selected record will be reviewed on the screen, as shown in Figure 3-14.

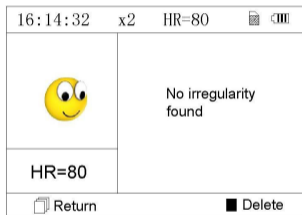


Figure 3-14

(1) “ Return”: press “ Return” button to return to record list screen.

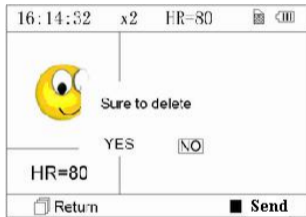


Figure 3-15

(2) “■ Delete”: press “■ OK” button to delete the selected current record, at this time it prompts that “Sure to delete?”, as shown in Figure 3-15. If the wireless transmission function is selected, here prompts “■ Send” instead. Press the “■

Send” button on the device, then on the host side, click the “Wireless Receive” icon while running the software “ECG Viewer Manager” on PC, the data will be transmitted and saved on PC.

4. Select “YES”, then press “■ OK” button to perform deletion and return to record list screen. Press “☐ Return” button directly, this record will not be deleted and the screen will return to record list screen.

3.4.2 Measurement Result Description

When device detects lead off or suspected arrhythmia ECG waveform, “Poor signal, Measure again (Figure 3-16)”, “Suspected a little slow beat (Figure 3-17)” etc. 17 types ECG measurement results can be obtained. Refer to Table 3-1 for details.

(1). Poor signal

(2).A little slow beat

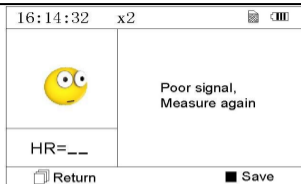


Figure 3-16

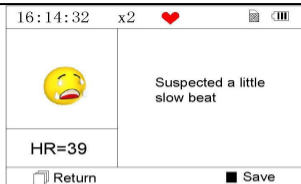
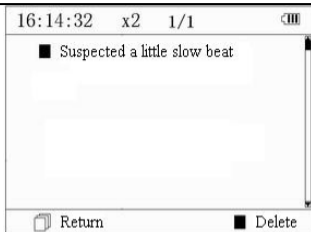
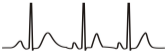


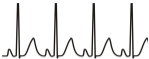
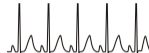
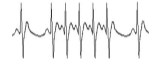



Figure 3-17

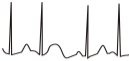

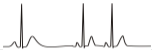
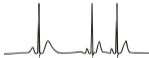


NOTE: Figure 3-16 and Figure 3-17 show the measurement results by continuous measuring method. The measurement by result continuous measuring method is shown on the left. If one screen hasn't enough space to show all the measurement results, the user can press Navigation Key " $\triangleleft/\blacktriangle$ " or " $\blacktriangledown/\blacktriangleright$ " to manually shift the screen upwards or afterwards



3.4.3 Measurement Result Table

No.	ECG Measurement Descriptions	Result	Waveform
1	No irregularity found		

2	Suspected a little fast beat	
3	Suspected fast beat	
4	Suspected short run of fast beat	
5	Suspected a little slow beat	
6	Suspected slow beat	
7	Suspected occasional short beat interval	

8	Suspected irregular beat interval	
9	Suspected fast beat with short beat interval	
10	Suspected slow beat with short beat interval	
11	Suspected slow beat with irregular beat interval	
12	Waveform baseline wander	
13	Suspected fast beat with baseline wander	


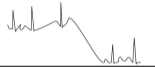



14	Suspected slow beat with baseline wander	
15	Suspected occasional short beat interval with baseline wander	
16	Suspected irregular beat interval with baseline wander	
17	Poor Signal, measure again	

Table 3-1

3.5 Data Management

3.5.1 Data Deletion

Select “Memory” on main menu screen, then press “ OK” button to enter memory screen, as shown in Figure 3-18.

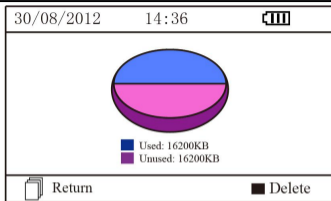



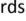


Figure 3-18 Memory Screen

Note: the used space plus the unused space equals to the total memory size (16MB/32MB).

“ **Return**”: press “ Return” button to return to main screen.

“ **Delete**”: press this key to delete all the records. Prior to deletion, a dialogue box “Sure to delete all?” will be presented on the screen, as shown in Figure 3-19. If choose “YES” and press “ OK” button, all the records will be deleted, as shown in Figure 3-20.

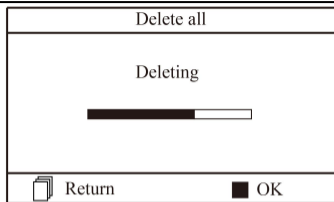


Figure 3-19

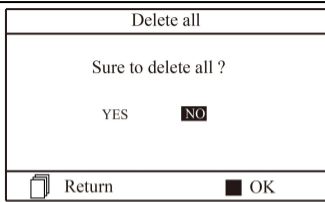


Figure 3-20 Records Deleting

Note: The operation of data deletion carried out in the memory screen means all the data in the device will be deleted; if you only want to delete one record, please do that on waveform review screen.

3.5.2 Upload Data

Connect the device to PC with USB data cable, the device will automatically enter the upload display as shown in Figure 3-21. The data can be dumped to PC for data management, review and analysis.

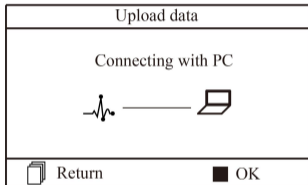


Figure 3-21 Upload Data Screen

NOTE: Ensure there is sufficient battery power when deleting or uploading the data since insufficient power may cause the interruption of operation.

3.6 System Setting

On the main menu screen, select “Setting”, then press “■ OK” button to enter system setting screen, as shown in Figure 3-22 and Figure 3-23

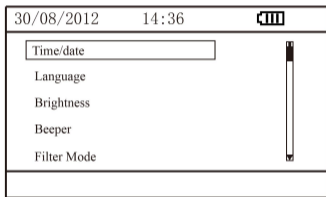


Figure 3-22

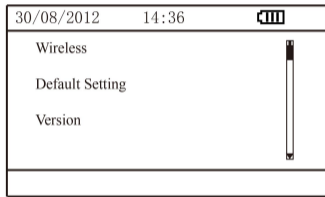


Figure 3-23

3.6.1 Time/Date Setting

1. On system setting screen, select “Time/date” with Navigation Key, then press “■ OK” button to enter time and date setting screen (date format: dd/mm/yyyy), as shown in Figure 3-24.
2. Use Navigation Key to adjust the value, and then press “■ OK” button to shift black cursor and adjust another value.
3. When finishing the settings, press “■ OK” button to save the setting and return to the upper level menu screen.

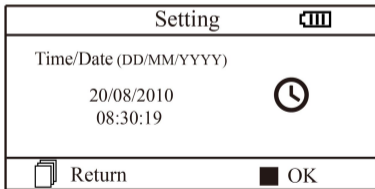


Figure 3-24 Time/date Setting

3.6.2 Language Setting

1. On system setting screen, select “Language” to enter language information screen, as shown in Figure 3-25.
2. Press the Navigation Key to select the language between “简体中文” and “English”. The default language is “简体中文”.
3. Press “■ OK” button to save setting and return to the upper level menu screen..

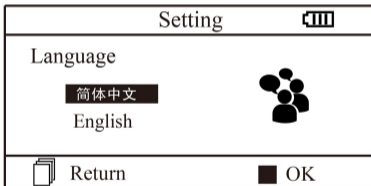


Figure 3-25 Language Setting

3.6.3 Brightness Setting

1. On system setting screen, select “Brightness” to enter brightness setting screen, as shown in Figure 3-26.
2. Adjust brightness with Navigation Key; 1~7 level adjustable;
3. Press “■ OK” button to save setting and return to the upper level menu screen.

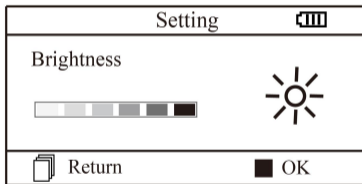



Figure 3-26 Brightness Setting

3.6.4 Beeper Setting

1. On system setting screen, select “Beeper” to enter beeper setting screen, as shown in Figure 3-27.
2. Choose “ON” or “OFF” with Navigation Key to turn on the beeper or turn off the beeper; the default setting is “ON”.
3. Press “ OK” button to save setting and return to the upper level menu screen.

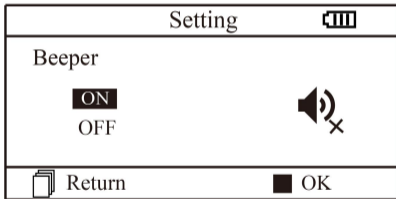


Figure 3-27 Beeper Setting

3.6.5 Measure Mode Setting

1. On system setting screen, select “Measure mode” to enter measure mode setting screen, as shown in Figure 3-28.

2. Choose “Normal” mode or “Enhanced” mode with Navigation Key. The default setting is “Normal” mode.

Normal mode: can filter the interference signal in ECG waveform.

Enhanced mode: can better reflect the fidelity of ECG waveform.

3. Press “■ OK” button to save setting and return to the upper level menu screen.

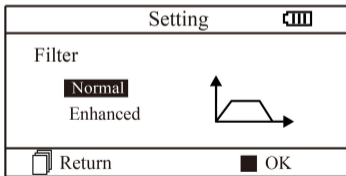


Figure 3-28 Measure Mode Setting

3.6.6 Wireless Setting

1. On system setting screen, select “Wireless” to enter wireless setting screen, as shown in Figure 3-29.
2. Press the Navigation Key to turn on or turn off the wireless function.
3. Press “■ OK” button to save setting and return to the upper level menu screen.

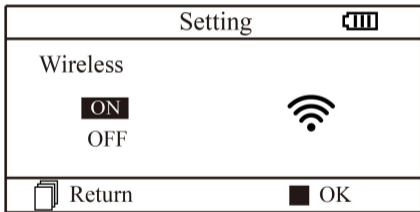


Figure 3-29 Wireless Selection

◆ **Downloading the APP software for smart phone**

The terminal devices such as smart phone can be used to receive data from the Easy ECG Monitor in real-time, and store the received data, review the stored data as well. You have to download the corresponding APP software on the smart phone.

For terminal devices with the Android system, please follow the procedure to download:

1. Install an APP software for scanning QR Code by smart phone, such as QuickMark, I-Nigma, Bee Tagg etc..
2. Run the APP software to scan the QR Code image in below figure, please focus the QR Code frame while scanning.
3. If successful scanned, then the scanning result that is a web link for downloading the APP




software "PC-80B. apk" will be displayed on the smart phone.

4. Open this web link by a web browser to download the APP software "PC-80B.apk". Install this APP software if successful downloaded.

For terminal devices with the iOS system (such as iPhone, iPad), please follow this procedure to download:

1. On the App Store of the device, enter "Shenzhen Creative" into the search line.

Note: if you use an iPad to search, please select "iPhone only" when searching.

2. Once the search results are listed, select the result with @health icon "", then download the corresponding APP software.

Instruction for Measurement

☞ Make sure the APP software is successful to connect with the Easy ECG Monitor.

☞ Refer to the manual of this APP software for detail operating.

✧ **Note: APP software for PC and Android smart phone is also available for download from below link:**

<http://www.creative-sz.com/software/>

3.6.7 Resume Default

1. On system setting screen, select “Default setting” to enter into default setting screen, as shown in Figure 3-29.
2. Choose “YES” or “NO” with Navigation Key. If the choice is “YES”, then press “■ OK” button, all the parameter settings will resume default settings.

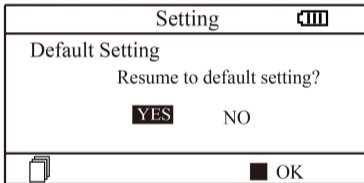



Figure 3-30 Default Setting

3. Press “ Return” button to back to upper level menu.

3.6.8 Version Info

On system setting screen, select “Version” to enter version information screen, as shown in Figure 3-30. “V52-10-10” is version number(refer to the actual version).

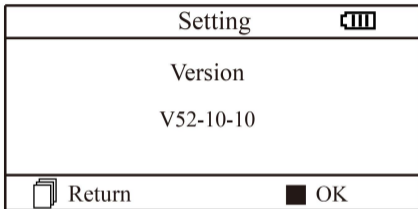


Figure 3-31 Version Information

3.7 Help

On main menu screen, select “Help”, then press “■ OK” button to enter help info screen. It provides different kinds of illustrations for ECG measuring methods. Refer to Section 3.2.1 for ECG measuring methods in details.

4 Technical Specifications

4.1 ECG Measurement

1. The number of measuring channel: Single channel ECG signal.
2. Electrodes: 3 embedded metal electrodes or 3 adhesive ECG electrodes connected to the connector of the device with lead wire.
3. Measurement modes: the detecting ECG signal can be detected by the following lead selection:
 - Lead I (between right hand and left hand);
 - Chest Lead (between right hand and chest);
 - Lead II (between right hand and left leg).
4. ECG bandwidth: 1Hz~40Hz
5. Internal noise level: $\leq 30\mu\text{Vp-p}$
6. Heart Rate measuring range: 30bpm~240bpm
7. Heart Rate measuring accuracy: $\pm 2\text{bpm}$ or $\pm 2\%$ whichever is greater

8. Display scale: 5.0mm/mV \pm 10%
9. Common-mode rejection ratio (CMRR): \geq 60dB
10. Waveform sweeping speed: 20mm/s \pm 10%
11. Input loop current: \leq 0.1 μ A
12. Input impedance: \geq 5 M Ω

4.2 Power

Power: 2 \times AAA alkaline batteries

Supply voltage range: 2.7~3.3VDC

Max average working current: \leq 50mA

Auto power-off: 35s \pm 5s (without operation)

Low battery indication: 2.5VDC \pm 0.2VDC

4.3 Classification

1. **The type of protection against electric shock:** Internally powered equipment
2. **The degree of protection against electric shock:** Type BF applied parts

3. **The degree of protection against harmful ingress of liquid:** Ordinary equipment without protection against ingress of liquid.
4. **Electro-magnetic Compatibility:** Group I, Class B

4.4 Display

1. LCD type: colour LCD
2. Display area: 57.6mm × 40mm

4.5 Data Memory

1. Up to 10-hour records can be saved in the built-in memory;
2. Stored data records can be uploaded to the computer by data cable provided by the manufacturer.

4.6 Overall Dimension and Weight

Dimension: (L) 125× (W) 70 × (H) 21.5 mm

Weight: 106g (not including batteries)

5 Accessories

A hanging cord

Two batteries (AAA)

A Pouch

A User Manual

A data cable (optional)

An ECG lead wire (optional)

Note: The accessories are subject to change. Please see the Packing List for detailed items and quantity.

6 Maintenance and Service

6.1 Maintenance

The life of this device is 5 years. In order to ensure its long service life, please pay attention to the maintenance.

1. Please change the batteries, when appears low-voltage icon.
2. It is recommended to wipe the metal electrode with medical alcohol, when the measuring signal is too weak.
3. Please wipe the surface of the unit with clean cloth or let it dry in air.
4. Please take out the batteries if the unit will not be used for a long time.

5. Storage and Transportation







Ambient temperature: -20 to 60°C

Relative humidity: 10~ 95%

Atmospheric pressure: 50~107.4Kpa

This device should be transported by land (vehicle or railway) or air in accordance with the contractual terms. Do not hit or drop it with force and protect it from sunlight and rain.

Notes

-  Keep it away from corrosive substances, explosive substances, high/low temperature and moisture.
-  If the device gets wet or has water steam, please stop operating it.
-  When it is taken from cold environment to warm and humid environment, please DO NOT use it immediately.
-  DO NOT operate the button on front panel with sharp materials.
-  DO NOT immerge the device into liquid. Do NOT spray any liquid on it directly.
-  The life of this unit is 5 years. In order to ensure its long service life, please pay attention to the use of maintenance.

6.2 Cleaning and Disinfecting Instruction

Surface-clean sensor with a soft gauze by saturating with a solution such as 75% isopropyl alcohol, if low-level disinfection is required, use a 1:10 bleach solution. Then surface-clean with a damp cloth and dry with a piece of cloth.

Caution: Do not sterilize by irradiation steam, or ethylene oxide.

7 Troubleshooting

✧Trouble

The device can not turn on.

Possible Reason	Solution
<ol style="list-style-type: none">1. The batteries are drained or almost drained.2. The batteries are not inserted properly.3. The device is broken out.	<ol style="list-style-type: none">1. Change batteries.2. Reinstall batteries.3. Please contact the local service center.

◆Trouble

The device is failure in measuring heart rate.

Possible Reason	Solution
<ol style="list-style-type: none">1. Electrode does not contact body well.2. Movement when measuring.3. Electromagnetic interference.4. Weak signal.	<ol style="list-style-type: none">1. Place the electrode correctly.2. When measuring, please keep quiet and avoid moving.3. Keep away from interference source.4. Change a place (chest) to measure.

◆ **Trouble**
















ECG waveform excursion, strong irrelevant waveform or the ECG waveform displays on the screen, but the prompt info is “Contact?”;




Possible Reason	Solution
<ol style="list-style-type: none">1. Skin is dry or oily.2. Electrode does not contact body tightly.3. Muscle strain.	<ol style="list-style-type: none">1. Clean skin with soap and water and eliminate skin crumb and grease to make the skin wet and greaseless.2. Press the electrode with a certain force.3. Keep hand relax while measuring.

8 Error Message and Solutions

Error	Reason	Solution
Communication error	The communication is interfered or abnormal	<ol style="list-style-type: none"><li data-bbox="701 242 1325 322">1. Press the "Return" button to exit from current status<li data-bbox="701 339 1286 419">2. Ensure the PC software can operate normally.<li data-bbox="701 437 1279 516">3. Restart the wireless function or the device.
Memory full	The memory of flash is full	<ol style="list-style-type: none"><li data-bbox="701 536 1358 573">1. Manually delete the ECG record directly;<li data-bbox="701 588 1339 625">2. Upload the record to PC, then delete it.

9 Key of Symbols

Symbol	Description	Symbol	Description
	Heart Rate (Unit: beats per minutes)		Battery indicator
	Measure mode: enhanced		Wireless transmission mark
	Full memory mark		
	Return button		Confirmation button
	Power button		Navigation key
	Type BF applied part		Data Interface Port
	Lead wire socket		Serial number
	Refer to User Manual		CE mark

	Manufacturer (including address and date)		USB data cable interface
EC REP	Authorised representative in the European community		Disposal of this device according to WEEE regulation.

Note: The symbols “”, “”, “”, “” and “” are displayed on the screen, the others are marked on the device panel.

Appendix

General Knowledge of ECG

I Normal sinus rhythm: In sinus conditions SA node paces the heart with the regularly rate and the normal rhythm. The

heart rate is in the range of 60 to 100 times per minute and the rhythm is regular. P wave is normal and each one is followed by a QRS wave. P-R interval: 0.12~0.20s;

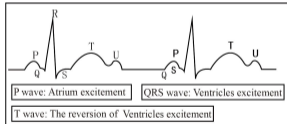


Figure 1 Normal ECG Waveform

QRS wave: 0.06~0.10s; No ectopic ECG activity.

Symptom: Sinus rhythm, heart rate: 60~100bpm

Indication: Normal

II Several abnormal ECG waveforms

- 1) **Tachycardia:** The heartbeat becomes faster. Tachycardia is determined by the rhythm of the heart, the average rhythm beats faster than 100 times a minute is considered as Tachycardia. The result description “Suspected fast beat” in this device may be doubted as Tachycardia.

Symptom: heart rate > 100bpm

Indication: It may occur with the normal people who have these physiology conditions: rage, fatigue, smoking, drinking too much wine, excessive coffee and strong tea, etc.

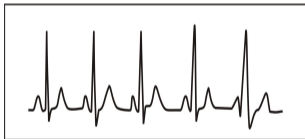


Figure 2 Tachycardia Waveform

Pathology: anemia, Hyperthyroidism, blood hypoxia, myocarditis, hypokalemia,

fever, influence of some medication (such as atropine, epinephrine etc.).

Suggestion: If it belongs to pathology condition, please go to hospital.

- 2) **Bradycardia:** The heartbeat becomes more slowly. It is determined by the rhythm of the heart, the average rhythm is less than 60 beats per minute is considered as Bradycardia. The result description “Suspected slow beat” in this device may be doubted as Bradycardia.

Symptom: heart rate < 60 bpm

Indication: It occurs when healthy people fall asleep, and it can be found in athlete (or those who love doing sports frequently), old people, or vagus excitement person.

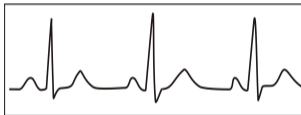


Figure 3 Bradycardia Waveform

Pathology: Sick sinus syndrome, Ischemic heart disease, Cardiomyopathy, intracranial hypertension, increased hypokalemia, Low temperature, period of convalescence of acute infectious disease or after use some medicines such as digitalis.

Suggestion: If it belongs to the pathology condition, please see the doctor.

- 3) **Premature beat:** In a normal beat a QRS waveform appears prematurely and is followed by a comeback pause. The result description “Suspected occasional short beat interval” in this device may be doubted as Premature beat.

What is premature beats? It is premature contraction. The normal heart beat is always regularly, and each beat interval is also symmetrically; Premature beat is out of this general regulation, the heart beats prematurely and followed by a long time interval; the phenomena of a premature beat between two heart beats is called inserting premature beat. The display of “early beat” may be suspected Premature beat.

According to different original positions, it can be divided into: Premature Atrial Contraction, Premature Nodal Contraction and Premature Ventricular Contraction (PVC), which needed to be judged by experts.

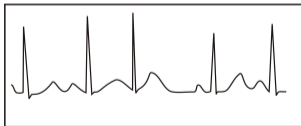


Figure 4 Premature Beat

Symptom: The heart beat is arrhythmic, it occurs as the phenomenon that the heart suddenly beats and then stops for a while. Some people have palpitation or have no symptom.

Indication: The premature beat can be found occasionally in healthy people, with no distinct symptom or sometimes with palpitation. This may be caused by fatigue, anxiety, insomnia, smoking too much, or drinking too much wine, coffee, strong tea etc. It can be recovered without treatment. However, if premature beat occurs frequently, continuously or in multi-focus, it will indicate functional cardiovascular disease. Please see a doctor as soon as possible.

Suggestion: The character of premature beat needs professional to confirm, so please save the temporal figure in time. When seeing a doctor , you can recall it to doctor to judge the character of premature (Premature Atrial Contraction, Premature Nodal Contraction, Premature Ventricular Contraction or multi-focus premature beat) and help cure.

Testee's Symptom: Normal heart beat is followed by a premature beat.

- 4) **Bigeminy:** It is a type of PVC in which a normal beat is coupled with a premature beat.

Indication: PVC occurs frequently.

Suggestion: Please see the doctor.



Figure 5 Bigeminy

- 5) **Trigeminy:** It is a type of PVC in which two normal beats are coupled with a premature beat.

Indication: PVC occurs frequently.

Suggestion: Please see the doctor.

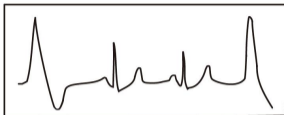


Figure 6 Trigeminy

The result description “Suspected irregular beat interval” in this device may be doubted as Bigeminy or Trigeminy.

- 6) **Short run of tachycardia:** PVC (Premature Ventricular Contraction) occurs more than 3 times continuously.

Testee's Symptom: PVC occurs More than 3 times continuously.

The heart beat is fast and regularly, but starts and stops suddenly.

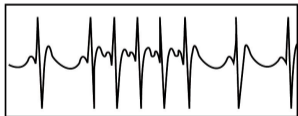


Figure 7 Tachycardia

According to the different of active original position, it can be divided into: Short Run, SVE Short Run (Needing professional to judge).

1)**Short Run** : It is caused by Premature Atrial Contraction or Nodal Premature Beat, frequency>180bpm.

Indication : Most commonly found in healthy people, it causes by deep respiration, tachypnea, positional changes, swallow, rage etc. It also appears in functional cardiac disease , such as Wolff - Parkinson-White Syndrome, rheumatic heart disease, coronary heart disease, Cardiomyopathy, Congenital heart disease, medicinal reaction (digitalis toxicosis) etc.

Suggestion: If it occurs time after time, please see the doctor as soon as possible.

2)**Ventricular Tachycardia** : Caused by Premature Ventricular Contraction, Heart Rate >140bpm.

Indication: Most commonly found in heart disease patient, it can cause ventricle fibrillation if it's serious, so the tester needs to see a doctor immediately.

Suggestion: The character of short run needs professional to confirm, so please save the temporal figure in time. And you can provide it to doctor as a reference.

Quality Certificate

Name: Easy ECG Monitor

Model: PC-80B

Date: _____

QA: _____

This product has been inspected in accordance with the standards specified in the User Manual.

Shenzhen Creative Industry Co., Ltd



Shenzhen Creative Industry Co.,Ltd.

**Floor 5, BLD 9, Baiwangxin High-Tech Industrial Park, Songbai Road,
Xili Street, Nanshan District, 518110 Shenzhen, P. R. China**

Tel: +86-755-2643 3514

Fax: +86-755-2643 0930

E-mail: info@creative-sz.com

Website: www.creative-sz.com

CE 0123



Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg Germany