



AIO Telehealth Cart

Unboxing Instructions for Visionflex Telehealth Cart

Cart Type	Ergotron SV40 with Drawer
PC Type	Lenovo All-In-One 24"
ProEX Hub	No
Patient-Facing Camera	Webcam
Patient-Facing Monitor	No
Document Version	1.0

Your Telehealth cart will arrive in a custom cardboard box, prewired and preconfigured. Depending on the selected configuration of the cart, some assembly will be required to mount the AIO Computer Screen onto the cart. The instructions below will guide you through this process, step by step.

Tools required: Scissors to cut packing straps and a knife to open the boxes.

1. Opening the Cart box.

Inspect the packaging and report any damage to Visionflex

Cut the strapping with scissors and stand the box up on the pallet end.



2. Remove the box.

Lift the box up and away from the base.

Remove the plastic bag from the cart.



3. Remove the monitor box.

Unclip the black strap to remove the monitor box from the packaging.



4. Lift the cart out of the base.

Fold down the rear cardboard flap.

Two people are required to lift the cart out of the base box by holding the rear and front handles.



5. **Checking the Cart once removed from the box.**

Once the cart has been removed from the box, inspect for damage, and report any findings to the Visionflex support team.

You will notice that the cart has been prewired.



6. **Mounting and connect the all-in-one computer / monitor.**

Remove the Lenovo box and remove the 4 mounting bolts from the rear side of the all-in-one.

Make sure that each thumb screw has the associated washer fitted.

Hold the monitor against the silver VESA mount on the cart and secure in place with the 4 thumb screws.

Connect the preconfigured cables to the underside of the monitor. These are the power cable, the HDMI connection for the patient-facing monitor and the USB feed from the 7-Port USB-Hub on the cart.



7. **Extend the patient-facing camera on a pole.**

Twist to loosen and tighten the tensioners and extend the pole that the webcam is connected to.

Position the webcam then tighten the screws on the side to stiffen the ball-joint. You should still be able to move the camera around to point it in the direction of your patient.



8. System Test

Turn on the cart battery by pressing the power button on the cart work-surface.



The computer and patient facing monitor can now be turned on. The battery life is indicated on the cart work surface. Use the spiral power cable at the bottom of the cart to charge the battery.



9. Congratulations

The support team at Visionflex look forward to working with you on the implementation of this new technology.

For more information or assistance with your cart assembly, please contact the support team by email on support@visionflex.com or by phone on 02 8914 4000.

