








# Primary Storage Drawer

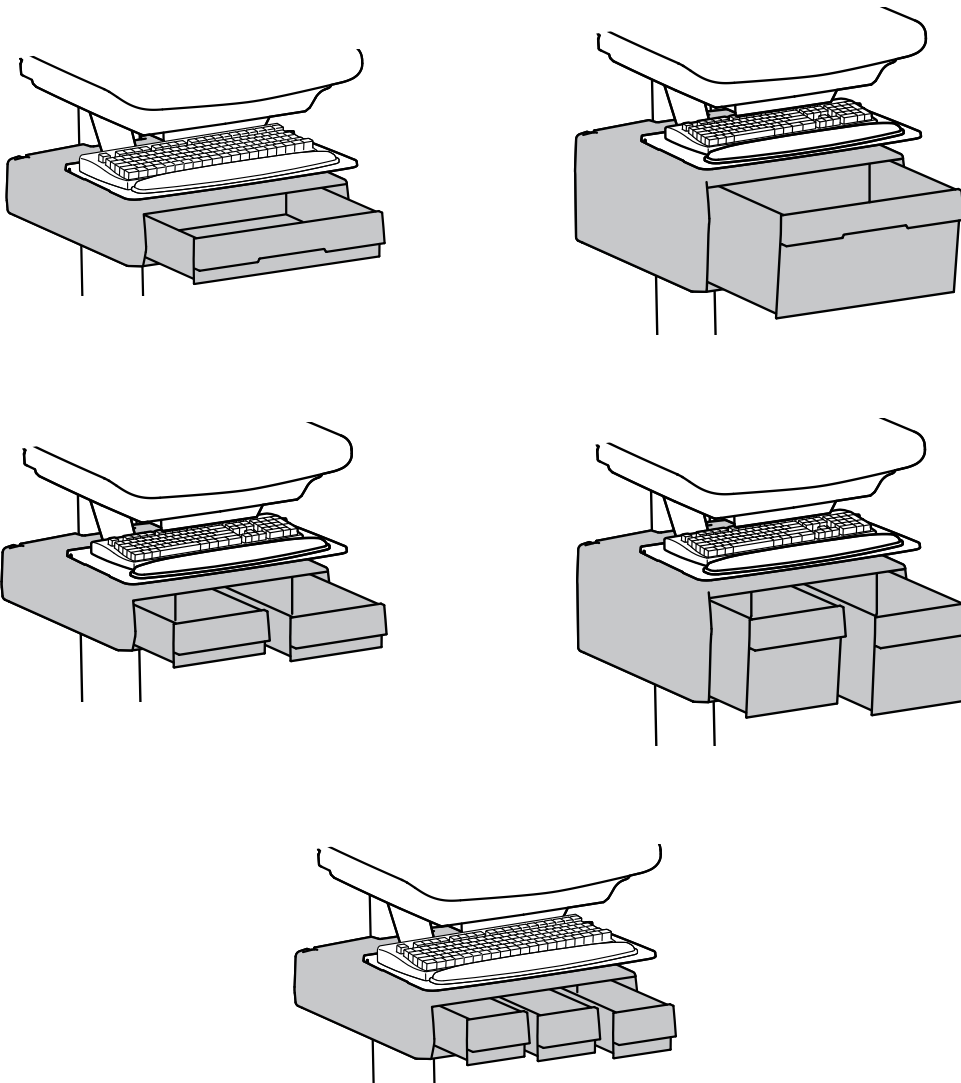


For more information and the latest Telehealth Cart User Guides,  
please visit: [info.visionflex.com/support-documents](http://info.visionflex.com/support-documents)

# Hazard Symbols Review

These symbols alert users of a safety condition that demands attention. All users should be able to recognize and understand the significance of the following Safety Hazards if encountered on the product or within the documentation. Children who are not able to recognize and respond appropriately to Safety Alerts should not use this product without adult supervision!

Symbol	Signal Word	Level of Hazard
	<b>NOTE</b>	A NOTE indicates important information that helps you make better use of this product.
	<b>CAUTION</b>	A CAUTION indicates either potential damage to hardware or loss of data and tells you how to avoid the problem.
	<b>WARNING</b>	A WARNING indicates either potential for property damage, personal injury, or death.
	<b>ELECTRICAL</b>	An Electrical indicates an impending electrical hazard which, if not avoided, may result in personal injury, fire and/or death.

## Safety

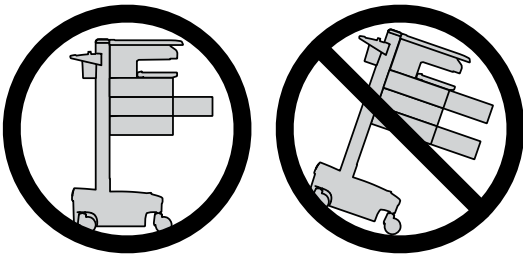

**CAUTION:** Close worksurface before opening drawers. Open only one drawer at a time. Do Not push cart when drawers or worksurface are open. Failure to follow these instructions may cause the cart to be unstable.

**! WARNING**



**IMPACT HAZARD!**  
MOVING PARTS CAN CRUSH AND CUT.

**Minimize Lift Tension BEFORE:**

- Removing Mounted Equipment.
- Shipping Cart
- Storing Cart

**To Minimize Lift Tension**


1. Lower worksurface to lowest position.
2. Turn adjustment nut at top of riser counterclockwise until it stops (Adjustment may require 40-60 revolutions).


**Failure to heed this warning may result in serious personal injury or property damage!**

For More information and instructions refer to product guide at: <http://support.ergotron.com> or contact Ergotron Customer Care at 1-800-888-8458.


822-052

# Features & Specifications


 CAUTION: DO NOT EXCEED MAXIMUM LISTED WEIGHT CAPACITY. SERIOUS INJURY OR PROPERTY DAMAGE MAY OCCUR!

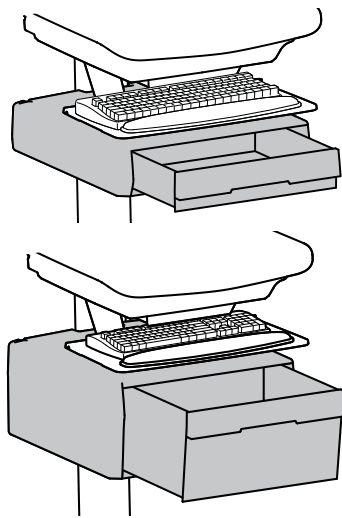






  
≤ 2 lbs (1 kg)  
Total weight capacity per drawer.

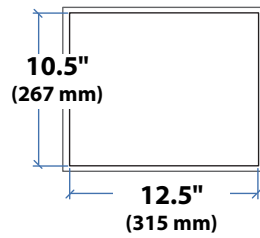


2.5"  
(64 mm)

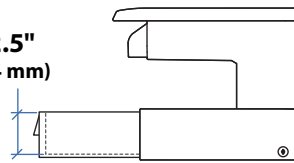


6"  
(152 mm)

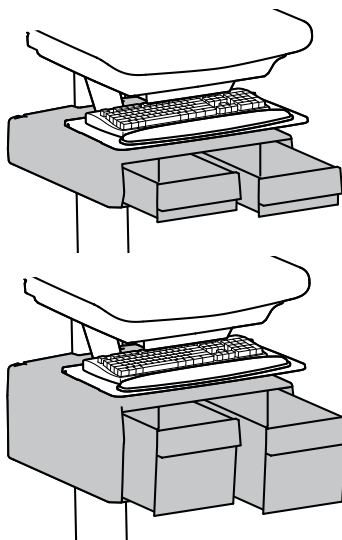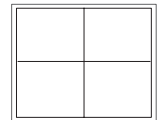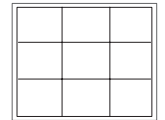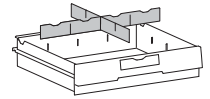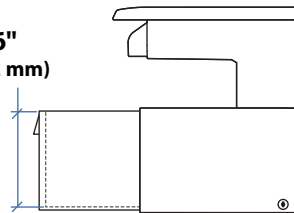






  
≤ 1 lbs (0.5 kg)  
Total weight capacity per drawer.

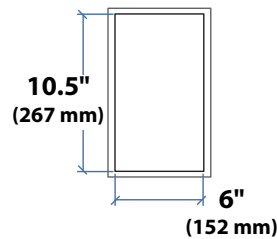





2.5"  
(64 mm)

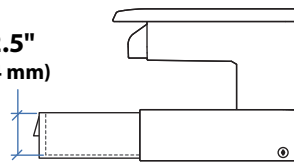



6"  
(152 mm)

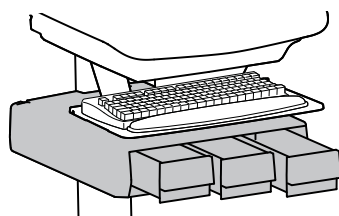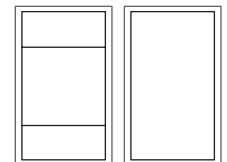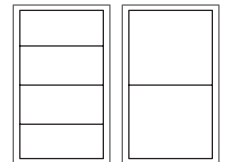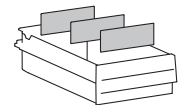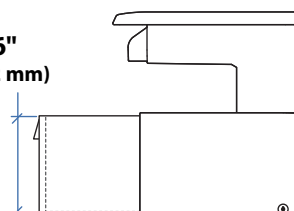

  
≤ 0.25 lbs (0.1 kg)  
Total weight capacity per drawer.

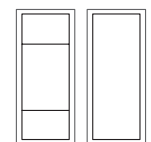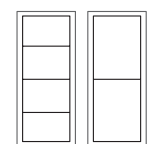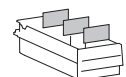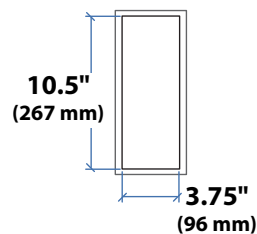


# Tools Needed

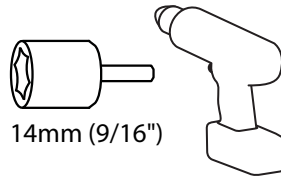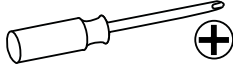

14mm (9/16")

# Components

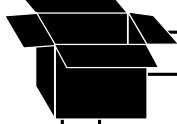


	A	B	C
1	<b>1x</b> 	<b>2x</b>  M6 x 43mm	<b>2x</b>  M6 x 15mm


This manual covers the installation of 5 possible drawers.  
There is only 1 drawer provided in this box.


	A	
2	<b>1 Drawer</b> <b>1x</b> 	<b>3x</b>  <b>3x</b> 
3	<b>1 Drawer</b> <b>1x</b> 	<b>2x</b>  <b>2x</b> 
4	<b>2 Drawers</b> <b>1x</b> 	<b>6x</b> 
5	<b>2 Drawers</b> <b>1x</b> 	<b>6x</b> 
6	<b>3 Drawers</b> <b>1x</b> 	<b>9x</b> 


# 1 Remove the top cover.

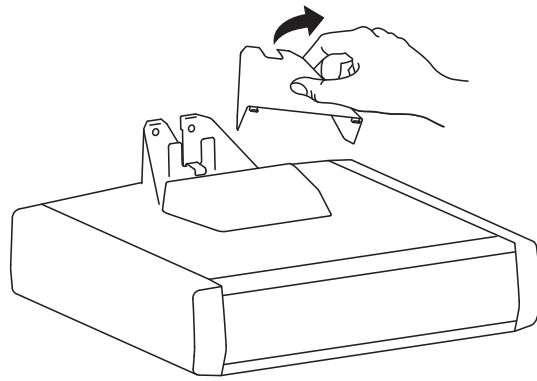



# 2 Hang the drawer tabs onto the brackets on the cart.

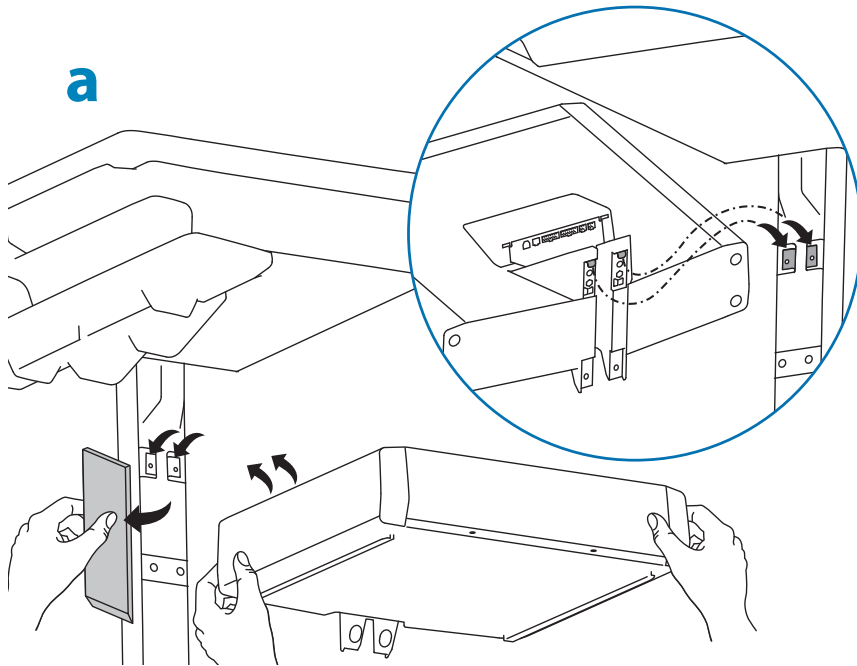


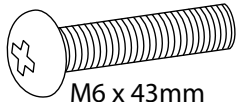
**b** 2x  M6 x 43mm

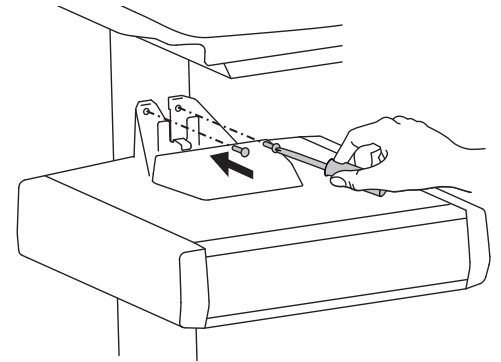


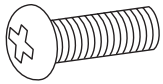
**c** 2x  M6 x 15mm

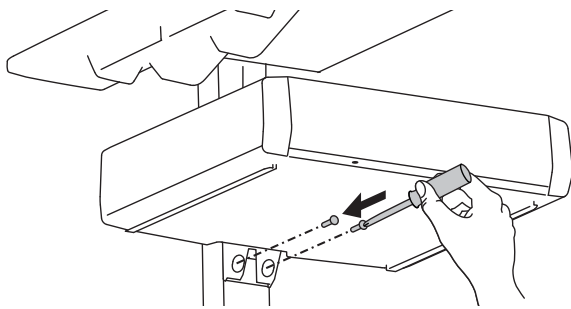



For Tall Drawers

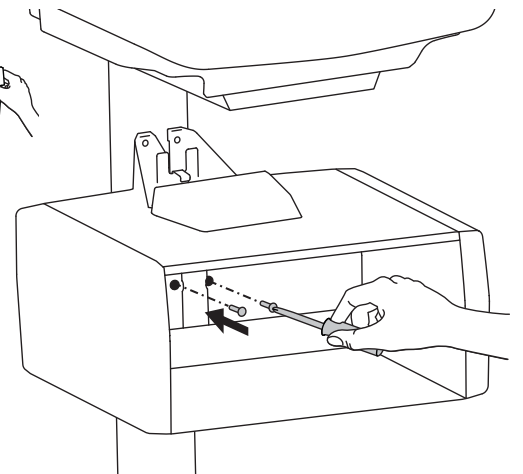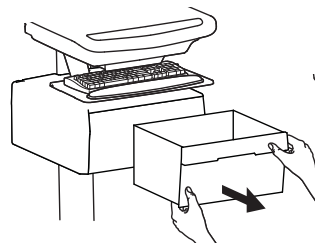

### 3 Replace top cover on drawer.

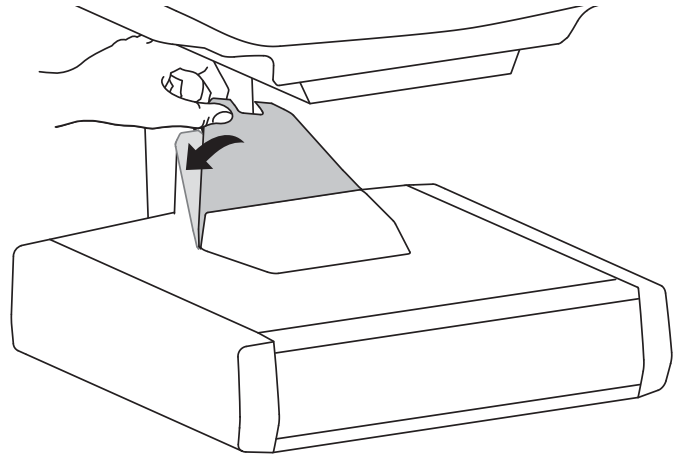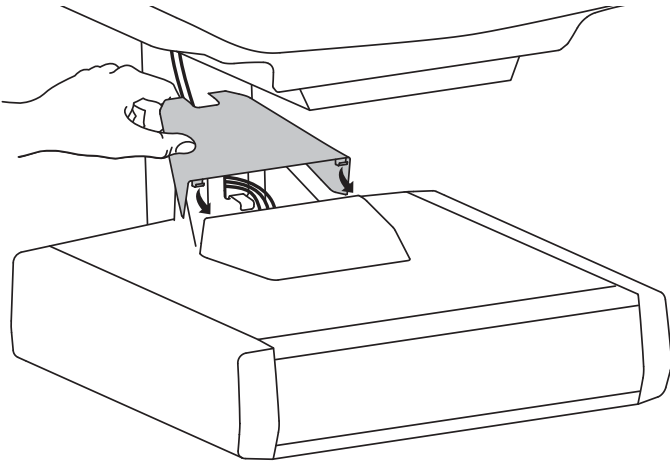

### 4 To unlock the storage drawer, turn key counter clockwise 1/4 turn.

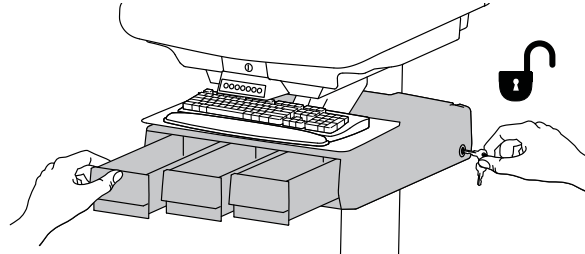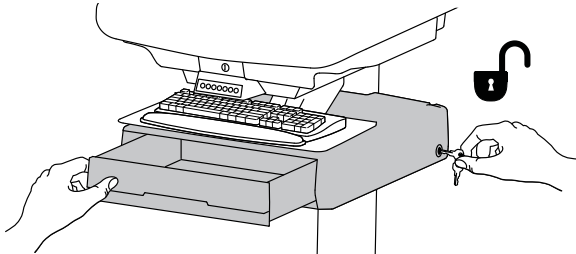

visionflex

For more information and the latest Telehealth Cart User Guides, please **scan the QR code** or visit: [info.visionflex.com/support-documents](https://info.visionflex.com/support-documents)

support@visionflex.com

+61 8914 4000

WWW.VISIONFLEX.COM

