

AIO Telehealth Cart

Unboxing Instructions for Visionflex Telehealth Cart

Cart Type	Ergotron Cart with Drawer
PC Type	All In One PC
ProEX Hub	No
Patient Camera	HD Webcam
Patient-Facing Monitor	Yes
Document Version	1.0

Your Telehealth cart will arrive in a custom cardboard box, prewired and preconfigured. Depending on the selected configuration of the cart, some assembly will be required to mount the All-In-One Computer Screen onto the cart. The instructions below will guide you through this process, step by step.

Tools required: Scissors to cut packing straps and a knife to open the boxes.

1. Opening the Cart box.

Inspect the packaging and report any damage to Visionflex

Cut the strapping with scissors and open the top of the cart box with a sharp knife.



2. Removing packaging and accessories from the box.

Open the Box and remove internal packaging material being careful to separate any accessories which may have been packed inside the cart box.



3. Tilting the Cart box upright

Once the packaging contents have been removed and all the accessories have been carefully separated to one side, the box can be tilted upright so that the cart can be wheeled out of the box.



4. Removing the Cart from the box and check it.

Wheel the cart from the box. The wheels may be locked so please check and unlock if necessary.

Once the cart has been removed from the box, inspect for damage, and report any findings to the Visionflex support team.

You will notice that the cart has been prewired.



5. Mounting and connect the all-in-one computer / monitor.

Remove the Lenovo box and remove the 4 mounting bolts from the rear side of the all-in-one PC.

Make sure that each thumb screw has the associated washer fitted.

Hold the monitor against the silver VESA mount on the cart and secure in place with the 4 thumb screws.

Connect the preconfigured cables to the underside of the monitor. These are the power cable, the HDMI connection for the patient-facing monitor and the USB feed from the USB-Hub on the work-surface.



6. Connecting the Patient Facing Display

Install the patient facing display monitor to the Swing Arm on the Cart.

Remove the thumb screw from the rear of the monitor.

The top hexagonal bolts slide into the mounting plate.

Use the provided spanner to fasten the two hexagonal bolts at the top and one thumb screw on the bottom.

Pull the chain at the hinge of the swing arm to unlock the hinge so that it can swing out.

Plug in the power and HDMI cables.



7. Extend the patient-facing camera on a pole.

Twist to loosen and tighten the tensioners and extend the pole that the webcam is connected to.

Position the webcam then tighten the screws on the side to stiffen the ball joint. You should still be able to move the camera around to point it in the direction of your patient.



8. System Test

Connect the cart to mains power and turn on the computer by pressing the power button on the right side of the device.

The username for the AIO computer is set to **“Telehealth Cart”** by default with no password but your IT department can change this as required.



Congratulations

The support team at Visionflex look forward to working with you on the implementation of this new technology.

For more information or assistance with your cart assembly, please contact the support team by email on support@visionflex.com or by phone on 02 8914 4000.

