



## Attaching the Accessory Holder to a Visionflex Telehealth Cart

Cart Type	Ergotron Cart
Accessory Holder	Clear Plastic Holder from Visionflex
Tools Required	Phillips-head screwdriver 7mm hex-socket screwdriver


1. **Attach the “C shaped” clear plastic piece (2) to the side of the Cart.**



2. **Fasten the flat clear plastic plate (1) with 2x M4x25 screws from inside the cart. Use the black plastic washers provided after inserting the screws.**



- 
3. **Mount the accessory holder and tighten the M4 nut along with the screws. Use a Phillips-head screwdriver and 7mm hex socket screwdriver as shown in the figure.**

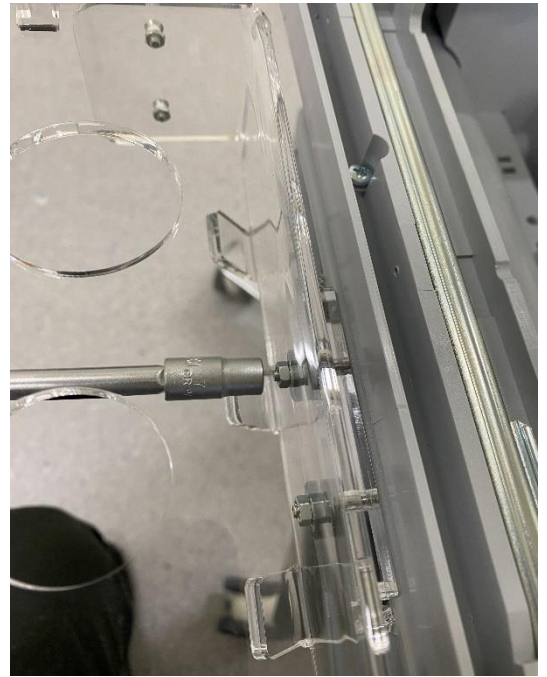

- 
4. **Check that the accessory holder doesn't make contact when the cart work-surface is slid open.**



---

For more guides, training videos and to request assistance visit: [www.visionflex.com/downloads-and-support](http://www.visionflex.com/downloads-and-support)

**visionflex**

After Sales and Technical Support:

+61 2 8914 4000  
support@visionflex.com